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Absorbent Non-Adherent Dressing (Including Lite)

Description

This Absorbent Non-Adherent Dressing helps wounds to heal with a specially designed absorbent pad which has a soft cushioning effect to protect the wound. The pad absorbs excess wound fluid which is important to allow your wound to heal. We have added an extra layer to make it easier and less painful when changing the dressing. Keep the dressing in place with a bandage, adhesive tape or our waterproof film covering.

Application of dressing:

1. Wash hands before applying the dressing.
2. Clean and dry the wound and surrounding skin gently.
3. Remove dressing from pouch.
4. Apply the dressing so that the shiny side of the dressing sits directly over the wound.

Change the dressing when you see blood/fluid seeping near the dressing edges.

Removing the dressing:

5. The dressing can be easily removed. Soaking the dressing with saline water where the wound is in contact can assist with removal if blood/fluid has dried.

Precautions for use

Seek medical advice immediately for all serious wounds and burns or if you experience redness or discomfort near the wound. Performance of product may vary if it is submerged under water for long periods.

Dressing is sterile unless the individual pouch is opened or damaged. Do not use the dressing if the pouch is open or damaged.

Single use. Store in a cool, dry place (<35 degrees Celsius).

Intended Use:

To absorb wound blood/exudate for minor burns, cuts, grazes while protecting the wound from external contamination.

Manufacturer:

BPositive Pte Ltd
www.bpositivehealthcare.com

Importer:

BPositive Pte Ltd
Singapore Business Federation Centre
160 Robinson Road, #14-04
068914 Singapore

BPositive Healthcare Pty Ltd

Suite 18, 11 Wilson Street, South Yarra VIC,
3141 Australia

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