



Singapore Business Federation Centre
160 Robinson Road, #14-04
068914 Singapore

Product Information Brochure

Bandage (Elastic Crepe) 888501891011003BARP

011003BA	Bandage (Elastic Crepe) 2.5cm x 1.8m
011005BA	Bandage (Elastic Crepe) 5cm x 1.8m
011075BA	Bandage (Elastic Crepe) 7.5cm x 1.8m
011010BA	Bandage (Elastic Crepe) 10cm x 1.8m
011015BA	Bandage (Elastic Crepe) 15cm x 1.8m

Bandage (Heavy Crepe) 888501891011205BASP

011205BA	Bandage (Heavy Crepe) 5cm x 1.8m
011275BA	Bandage (Heavy Crepe) 7.5cm x 1.8m
011210BA	Bandage (Heavy Crepe) 10cm x 1.8m
011215BA	Bandage (Heavy Crepe) 15cm x 1.8m

Bandage (Conforming Gauze) 888501891011103BAS2

011103BA	Bandage (Conforming Gauze) 2.5cm x 2mm
011105BA	Bandage (Conforming Gauze) 5cm x 2m
011175BA	Bandage (Conforming Gauze) 7.5cm x 2m
011110BA	Bandage (Conforming Gauze) 10cm x 2m
011115BA	Bandage (Conforming Gauze) 15cm x 2m

Intended Use

To be used for retaining a wound dressing and provide light support and compression.

Bandage (Elastic Crepe) - This bandage is soft and conformable to allow joint movement. Good for retaining dressings, light support and compression. Non-sterile and latex free.

Bandage (Conforming Gauze) - This bandage is highly elastic, soft and conformable to allow joint movement. The lightweight construction provides good breathability and air circulation. Good for retaining dressings, light support and compression. Non-sterile and latex free.

Bandage (Heavy Crepe) - This bandage is ideal for securing a wound dressing, supporting strains, and providing compression to alleviate swelling. Non-sterile and latex free

Warnings

Avoid applying too tight on limbs as it can constrict blood flow and lead to injury.

This product may not be right for you. Please consult a healthcare professional especially if symptoms worsen or change unexpectedly.

Non-sterile

How to use

1. Wash your hands before and after use . Gently clean the skin, and dry thoroughly.
2. Wrap the bandage in a spiral around the dressing or device, overlapping half of the bandage each time.
3. Ensure only minimal tension has been used; just enough to keep the dressing or device in place.