



Singapore Business Federation Centre
160 Robinson Road, #14-04
068914 Singapore

Product Information Brochure

Gauze Swabs (Non-sterile) 888501891010875BAVV

010875BA Gauze Swabs 7.5 cm (8PLY)
010810BA Gauze Swabs 10 cm (8PLY)
010805BA Gauze Swabs 5 cm (8PLY)

Gauze Swabs (Sterile) 888501891010975BAW

010975XA Sterile Gauze Swabs 7.5 x 7.5 cm (PK/5)
010975YA Sterile Gauze Swabs 7.5 x 7.5 cm (PK/3)
010910BA Sterile Gauze Swabs 10 x 10 cm (PK/5)
010975DA Sterile Gauze Swabs 7.5 x 7.5 cm (Dispenser - PK/5)

Intended Use

This soft and highly absorbent gauze can be used to clean a wound before it is dressed. Is available in sterile and non-sterile versions

Warnings

Single use application.

How to use

Remove from pack using aseptic technique, typically with forceps. Dampen the gauze in saline water. Clean the wound area with the dampened gauze using forceps. Recommended to clean in one direction discard. Clean the remaining areas with a clean dampened gauze using this one use and throw technique to minimise cross contaminating the area.