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bpositive

bpositive

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Soft & Flexible Dressing + Pad

Description

This dressing is extremely comfortable and conforms to your skin as it is made from a soft medical grade fabric. It has a specially designed absorbent pad which has a cushioning effect to protect the wound. The pad absorbs excess wound fluid which is important to allow your wound to heal. The nature of this dressing makes it ideal for most areas of the body, particularly near joints or areas of high movement.

Application of dressing:

1. Wash hands before applying the dressing.
2. Clean and dry the wound and surrounding skin gently.
3. Remove dressing from pouch.
4. Remove first half of the white backing paper.
5. Apply the dressing so that the white pad sits directly over the wound.
6. Remove the second white backing paper and press lightly down to secure the film to the skin.

Change the dressing when the fabric is not completely sticking to the skin or blood/fluid seeps through the pad.

Removing the dressing:

7. As this is an adhesive product, carefully remove the dressing, especially if skin is sensitive.

Precautions for use

Seek medical advice immediately for all serious wounds and burns or if you experience redness or discomfort near the wound. Performance of product may vary if it is submerged under water for long periods.

Dressing is sterile unless the individual pouch is opened or damaged. Do not use the dressing if the pouch is open or damaged.

Single use. Store in a cool, dry place (<35 degrees Celsius).

Intended Use:

To absorb exudate and protect the wound from further trauma.

Basic UDI-DI - 888501891010601BAT2

Manufacturer:

BPositive Pte Ltd
www.bpositivehealthcare.com

Importer:

BPositive Pte Ltd
Singapore Business Federation Centre
160 Robinson Road, #14-04
068914 Singapore

BPositive Healthcare Pty Ltd

Suite 18, 11 Wilson Street, South Yarra VIC,
3141 Australia

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