



Singapore Business Federation Centre
160 Robinson Road, #14-04
068914 Singapore

Product Information Brochure

Soft & Flexible Roll 888501891010703BATP
010703BA Soft & Flexible Roll 2.5cm x 10m
010705BA Soft & Flexible Roll 5cm x 10m
010705SA Soft & Flexible Roll 5cm x 10m (Tan)
010710BA Soft & Flexible Roll 10cm x 10m
010710SA Soft & Flexible Roll 10cm x 10m (Tan)
010715BA Soft & Flexible Roll 15cm x 10m

Intended Use

To hold or secure things like wound dressings to the skin or provide support to joint movement.

Warnings

Non-sterile

This product may not be right for you if your skin becomes red, itchy or irritated. Gently remove and consult your healthcare professional

If symptoms worsen or change unexpectedly, talk to your healthcare professional.

How to use

1. Carefully cut the length of tape required
2. Gently pull the sides of the tape to separate the backing paper
3. Peel off the backing paper and secure in place