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(Ver01) 35°C / 95°F

## Water & Bacteria Proof Film Dressing + Pad

### Description

This dressing helps wounds heal with a specially designed absorbent pad which has a soft cushioning effect to protect the wound. The pad has an extra layer which minimises pain when removing the dressing. In addition, our water and bacteria-proof protective film helps reduce the risk of infection. As it is waterproof, you can shower with it. This dressing provides a mechanical barrier to water and bacteria. It does not contain bactericidal agents.

### Application of dressing:

1. Wash hands before applying the dressing.
2. Clean and dry the wound and surrounding skin gently.
3. Remove dressing from pouch.
4. Remove first half of the white backing paper.
5. Apply the dressing so the white pad sits directly over the wound.
6. Remove the second white backing paper and press lightly down to secure the film to the skin.
7. Remove the top carrier where the orange arrow points.

Change the dressing when the film is not completely sticking to the skin, or you see blood/fluid seeps through the pad.

### Removing the dressing:

As this is an adhesive product, carefully apply and remove the dressing especially if skin is sensitive. Gently lift one corner and stretch the clear film outwards from the pad. Continue this around the pad until dressing is removed.

### Precautions for use

Seek medical advice immediately for all serious wounds and burns or if you experience redness or discomfort near the wound. Performance of product may vary if it is submerged under water for long periods.

Dressing is sterile unless the individual pouch is opened or damaged. Do not use the dressing if the pouch is open or damaged.

Single use. Store in a cool, dry place (<35 degrees Celsius).

### Intended Use:

To absorb exudate and reduce the risk of infection at the applied wound area for minor wounds by providing a mechanical barrier to water and bacteria.

Basic UDI-DI - 888501891020304BASZ

### Manufacturer:

BPositive Pte Ltd  
www.bpositivehealthcare.com

### Importer:

BPositive Pte Ltd  
Singapore Business Federation Centre  
160 Robinson Road, #14-04  
068914 Singapore

### BPositive Healthcare Pty Ltd

Suite 18, 11 Wilson Street, South Yarra VIC,  
3141 Australia

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