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(Ver01) 35°C / 95°F

Water & Bacteria Proof Film Dressing**Description**

This transparent dressing helps wounds heal by acting as a protective barrier to germs and contaminants. In doing so, it helps reduce the risk of infection at the wound site. It is ideal for minor wounds like burns, cuts, grazes and covering sutures. As it is very conformable, you can use it for difficult areas like fingers, toes or even on the face. If your wound is still releasing fluid, you will need to apply an absorbent dressing directly over the wound and then cover it with this dressing. This dressing provides a mechanical barrier to water and bacteria. It does not contain bactericidal agents.

Application of dressing:

1. Wash hands before applying the dressing.
2. Clean and dry the wound and surrounding skin gently.
3. Remove dressing from pouch.
4. Remove first half of the white backing paper.
5. Apply the dressing directly over the wound.
6. Remove the second white backing paper and press lightly down to secure the film to the skin.
7. Remove the top paper carrier frame leaving just the film on your skin.

Change the dressing when the film is not completely sticking to the skin.

Removing the dressing:

8. As this is an adhesive product, carefully apply and remove the dressing, especially if your skin is sensitive. Gently lift one corner and stretch the clear film outwards. Continue stretching around the dressing until it is removed.

Precautions for use

Seek medical advice immediately for all serious wounds and burns or if you experience redness or discomfort near the wound. Performance of product may vary if it is submerged under water for long periods.

Dressing is sterile unless the individual pouch is opened or damaged. Do not use the dressing if the pouch is open or damaged.

Single use. Store in a cool, dry place (<35 degrees Celsius).

Intended Use:

To protect and reduce the risk of infection at the applied wound area for minor wounds by providing a mechanical barrier to water and bacteria.

Basic UDI-DI - 888501891020406BATN

Manufacturer:
BPositive Pte Ltd
www.bpositivehealthcare.com

Importer:
BPositive Pte Ltd
Singapore Business Federation Centre
160 Robinson Road, #14-04
068914 Singapore

BPositive Healthcare Pty Ltd
Suite 18, 11 Wilson Street, South Yarra VIC,
3141 Australia

Local Authorised Representative:
T T MEDICAL MANAGEMENT SDN. BHD.
A-3A-03 EkoCheras, No:693, Batu 5 Jalan Cheras,
56000 Kuala Lumpur, Malaysia
Tel : +603 9134 5391

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