## Instruction For Use – Water & Bacteria Proof Film Roll

**Product description/composition:** This water & bacteria proof roll helps to protect the wound and reduce the risk of infection by providing a mechanical barrier. The dressing does not contain bactericidal agents and is therefore non-antibacterial. As it is waterproof, you can shower with it. It's made of high quality non-woven fabric or high permeable PU film coated with medical degree acrylic adhesive and release paper or release film.

**Intended use:** Used to protect and reduce the risk of infection at the applied wound area for minor wounds, such as cuts, grazes and stitches.

**Contraindication:** Allergies to any component of the plaster.

### Caution and warning:

This product may not be right for you if your skin becomes red, itchy or irritated. Gently remove and consult your healthcare professional. If symptoms worsen or change unexpectedly, talk to your healthcare professional.

### How to use:

1. Wash your hands before and after dealing with a wound. Gently clean the wound and surrounding skin, and dry thoroughly.
2. Cut the required length from the roll. Fold along the length of the film to crack the protective backing paper.
3. Remove the backing paper - Apply the sticky side to desired area and gently smooth down.
4. Remove the blue-coloured grid covering and gently smooth down again.

To remove

5. To remove, pull the film to gently stretch it parallel to the skin and lift off.

### Storage and transportation:

Store in a cool, dry place (<35 degrees Celsius).

### Shelf life:

3 years. See sales package for the production date and expiration date of the product.

**BPOSITIVE PTE LTD,**  
Singapore Business Federation Centre,  
160 Robinson Road, #14-04, 068914 Singapore

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**BPOSITIVE PTE LTD,**  
Singapore Business Federation Centre,  
160 Robinson Road, #14-04, 068914 Singapore

(+65 3135 1111)  
hello@bpositivehealthcare.com

UDI-DI: 888501891020505BATU

**Manufacturer / Pengilang:**

BPositive Pte. Ltd.

**Address / Alamat:** Singapore Business Federation Centre 160 Robinson Road, #14-04,  
068914 Singapore

**Tel:** +65 6883 0688

**Email:** [sales@bpositivehealthcare.com](mailto:sales@bpositivehealthcare.com)

**Label identification / Pengenalan Label:**

	Batch code <i>Kod kelompok</i>		Please don't reuse it. <i>Jangan gunakan semula.</i>
	Please read the instructions for use carefully before use. <i>Sila baca arahan penggunaan dengan teliti sebelum ini.</i>		Medical Device <i>Peralatan Perubatan</i>
	Warning and attention, please refer to the instructions for use. <i>Amaran dan perhatian, sila rujuk manual penggunaan.</i>		Manufacturer <i>Pengilang</i>

Revision No. 3rd: March 2024

**BPOSITIVE PTE LTD,**  
Singapore Business Federation Centre,  
160 Robinson Road, #14-04, 068914 Singapore

(+65 3135 1111)  
[hello@bpositivehealthcare.com](mailto:hello@bpositivehealthcare.com)